

SHENYUN

SHENYUN

Male External Catheter





PRODUCT TRADING

Male External Catheter

Male External Urinary Sheath, a self-adhesive latex-free silicone urinary sheath, is a good alternative to indwelling urinary catheters, diapers or other absorbent products. The Urinary Sheath is designed to be leak-free and comfortable, providing a reliable solution for draining urine from the bladder. It is designed to be a safe and leak-free treatment for male urinary incontinence.

■ Features

- Good elasticity and tensile strength for good adhesion.
- Built-in adhesive strips increase patient comfort and ease of use.
- Silicone (latex-free) eliminates skin irritation.
- Transparency: Clear cover for easy monitoring of skin conditions.
- Available in a variety of sizes to suit different ages and body types.


■ Precautions

- Keep the genital area dry and clean to avoid infection.
- Make sure you choose the right size sheath, not too tight or too loose.
- Change the sheath and bag regularly to keep them clean and hygienic.
- Check your skin regularly to see if there is any redness, swelling or damage.
- Make sure the bag is lower than the bladder to prevent urine from backing up or overflowing.



■ Clinical application and effect

Urine sheaths play an important role in various clinical scenarios, providing a convenient and effective urine management solution suitable for a variety of patients and medical needs.

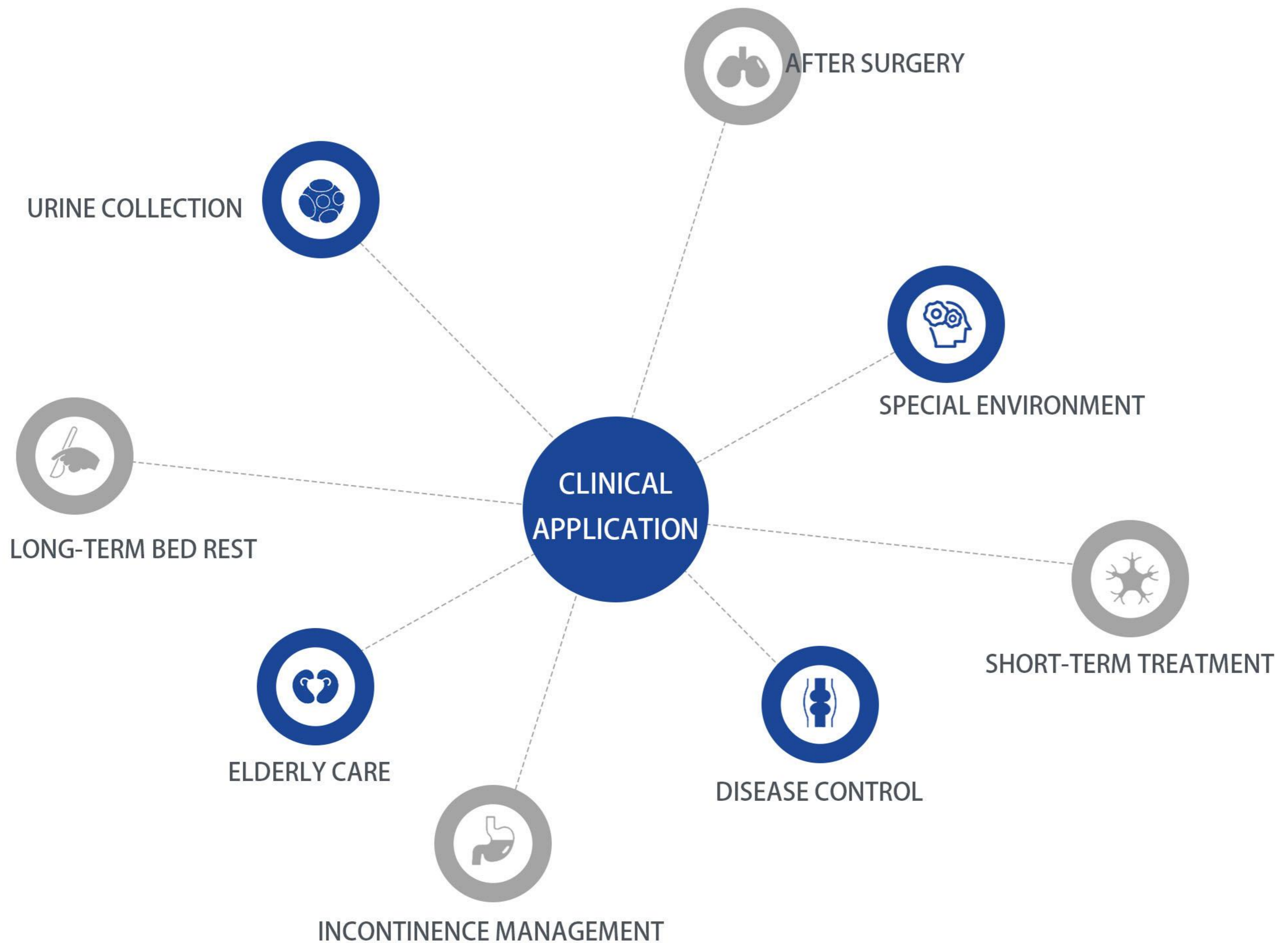

Urine collection

For patients who require short-term or temporary urine collection, such as before and after recovery from surgery.

Long-term bed rest

Provides a convenient way to manage urine for patients who are bedridden or have limited mobility.

Elderly Care

Urine sheaths are often used in elderly patients to facilitate daily urine collection and management.

incontinence

Urine sheaths can be used as a way to manage incontinence issues by reducing the chances of urine leaking accidentally.

Disease control

For example, urinary system diseases or neurological diseases, urinary sheaths can help control urine excretion.

Short-term treatment

For treatments that require temporary restriction of urine output, such as during radiation therapy.

After surgery

Urine sheaths can reduce the workload of patients and caregivers and ensure effective management of urine output.

Special Environment

In wilderness rescue or emergency situations, urine sheaths can be used as a temporary method of urine management.

ADVANTAGES COMPARISON

Advantages of urinary sheaths over indwelling urinary catheters

Easy to wear and use

- Urine sheath: It does not need to be inserted into the urethra or bladder, it only needs to be attached to the penis externally, and it is relatively simple and quick to use and replace.
- Indwelling urinary catheter: It needs to be inserted into the urethra into the bladder by a professional, and the operation is more complicated and requires special technology and equipment.

Comfort


- Urine sheath: Made of soft material, it is comfortable to wear and avoids the discomfort that may be caused by direct insertion into the urethra and bladder.
- Indwelling catheter: Indwelling catheter may cause irritation or discomfort of the urethra and bladder, especially when used for a long time.

Reduced risk of infection


- Urine sheath: The external fit design of the urine sheath can effectively reduce the risk of urine leakage and infection, especially for patients with short-term use.
- Indwelling catheter: The risk of urinary tract infection may increase during and after catheterization, requiring strict disinfection and care measures.




OPERATION PROCESS

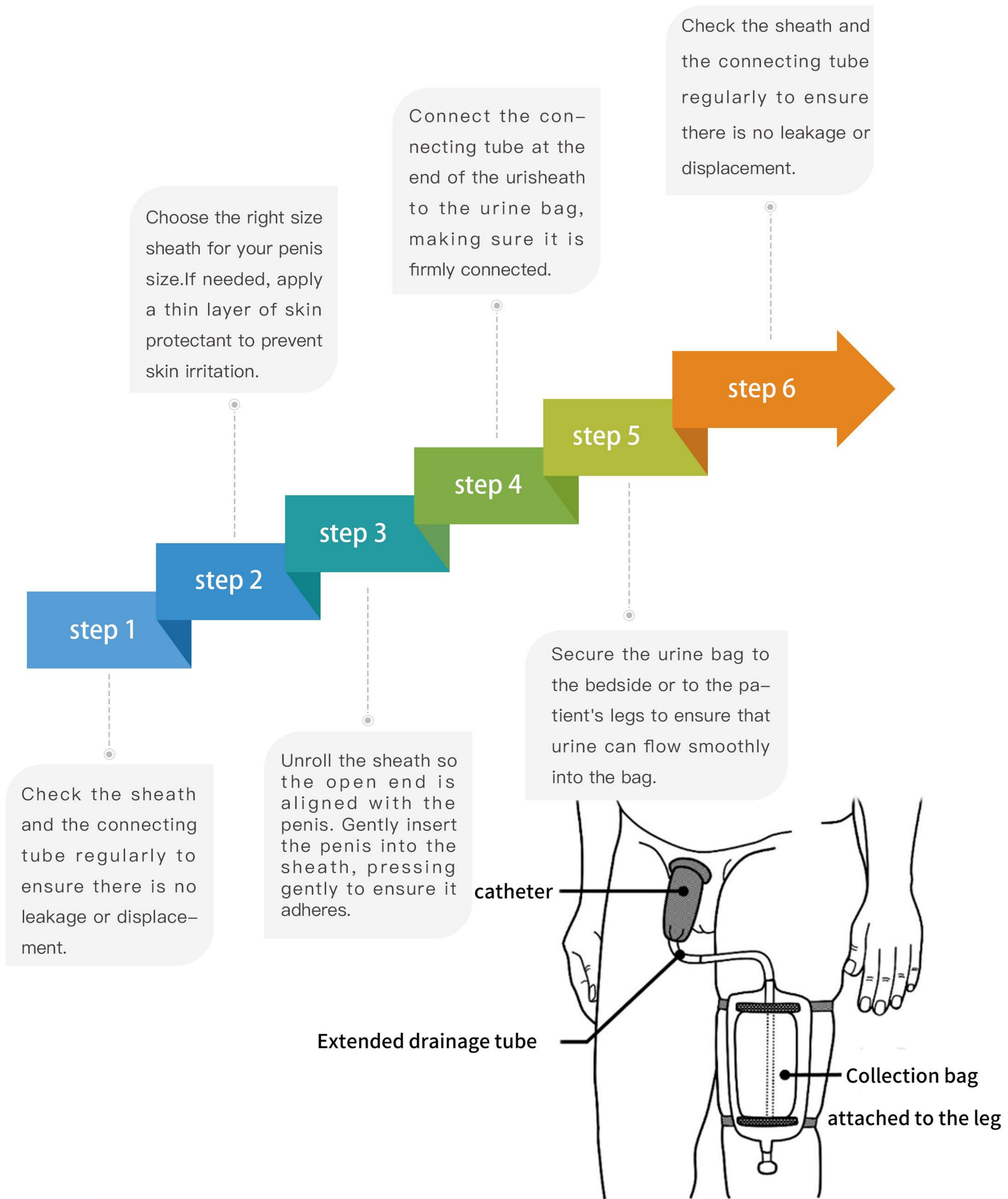


■ **STANDARD MODEL**

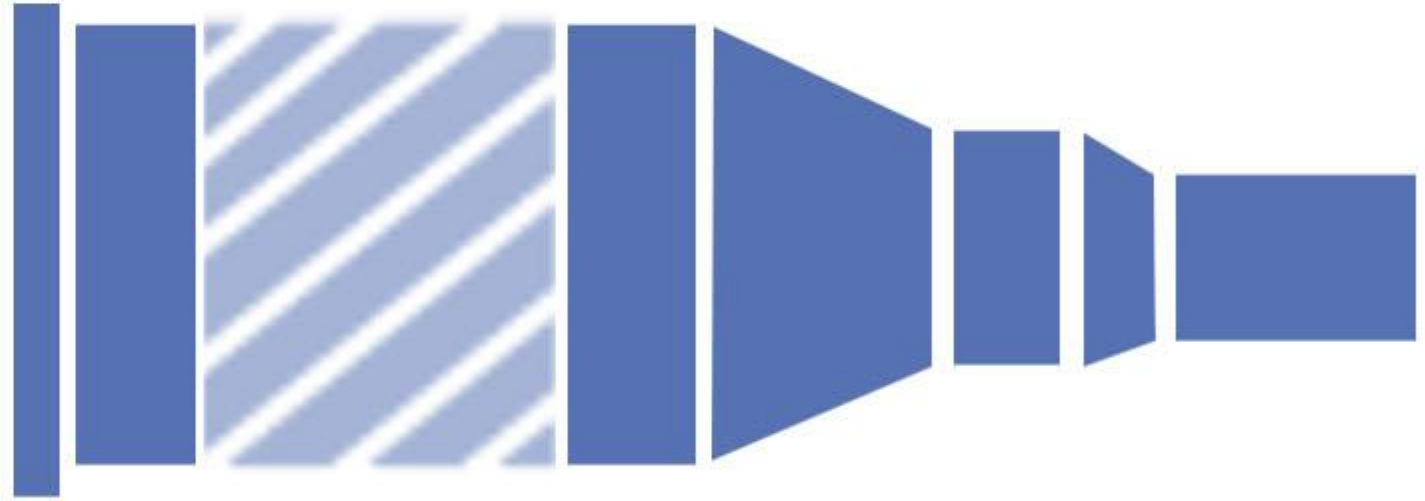model	size	image
SY08-01	18mm	
SY08-02	20mm	
SY08-03	25mm	
SY08-04	29mm	
SY08-05	32mm	
SY08-06	36mm	
SY08-07	41mm	

■ **SHORT MODEL**

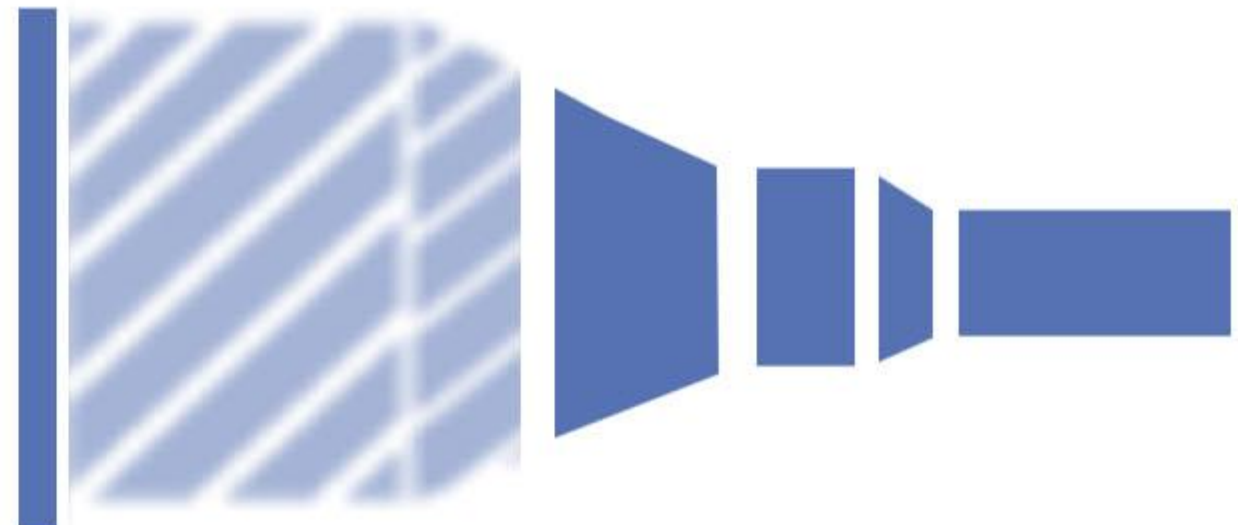
model	size	image
SY08-08	18mm	
SY08-09	20mm	
SY08-10	25mm	
SY08-11	29mm	
SY08-12	32mm	
SY08-13	36mm	
SY08-14	41mm	

■ **LENGTHENED MODEL**

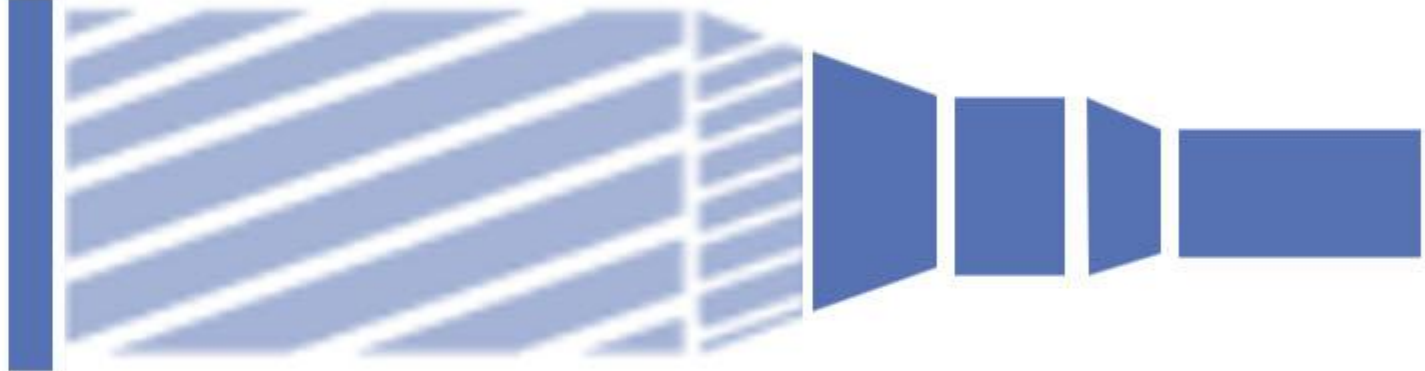model	size	image
SY08-15	18mm	
SY08-16	20mm	
SY08-17	25mm	
SY08-18	29mm	
SY08-19	32mm	
SY08-20	36mm	
SY08-21	41mm	

■ PACKING SPECIFICATIONS

Inner box quantity	Inner box size	Quantity in outer box	Carton size
30pcs	273*200*105 mm	360pcs	565*415*440mm

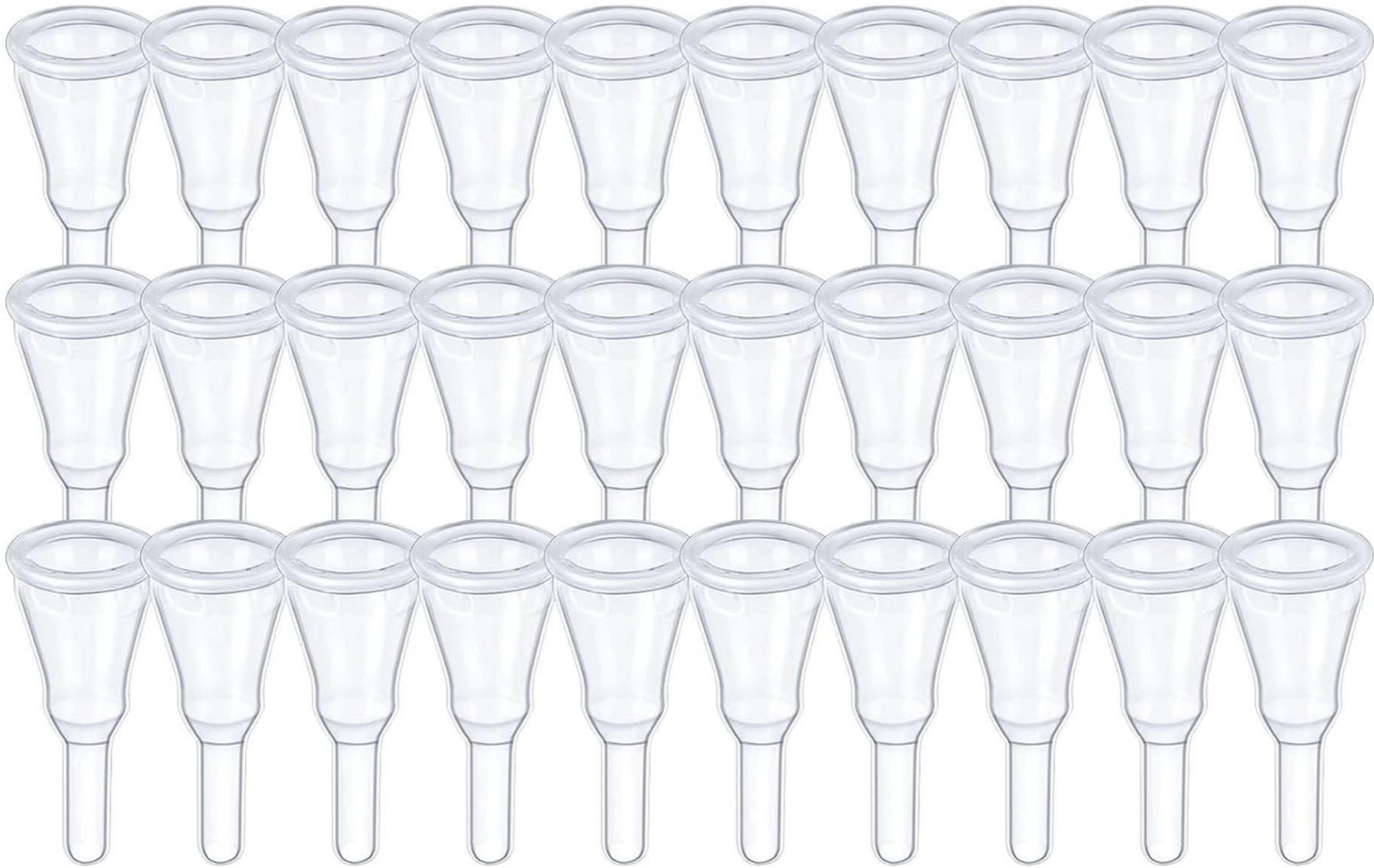

WE CARE, WE CAN

Suzhou Shenyun Medical Equipment Co., Ltd.
Add: No. 666, Yinzang Road, Linhu Town,, Suzhou City 215104, China
P: + 86 182 2153 2164 | E: Jolin@symedical.com.cn
www.shenyunmedical.com



Jack Liu



@shenyun_medical



@shenyun_medical Suzhou Shenyun Medical Equipment Co., Ltd.

